

## Materials Needed

- Distilled or purified water (avoid tap water to prevent clogging)
- Alcohol wipes
- Clean paper towel
- Teaspoon

## 1. Empty Medication

- Pour out any remaining medication.

## 2. Add Water & Shake

- Add 1 teaspoon of distilled or purified water to the atomizing cup.
- Shake for 5 seconds with the lid closed.



## 3. Wipe Clean

- Pour out the water.
- Wipe the inside of the atomizing cup with an alcohol wipe.
- Add 1 teaspoon of clean water, close the lid.

## 4. Activate Cleaning Cycle

- Press and hold the power button until it flashes green and blue.
- Allow it to sit for 2 minutes during the cleaning cycle.

## 5. Finish

- Pour out the water and let the atomizing cup dry upside down on a clean paper towel.

## 6. Maintenance

- After each use, repeat the cleaning process.

**You are always welcome to refer patients back to Conversio Health and connect with a team member 7 days a week at (866) 239-3784.**

## Do you have any questions for our care team?



### Call Conversio Health at:

(866) 239-3784

Hours of Operations:

6:00 am to 5:00 pm Pacific Time

9:00 am to 8:00 pm Eastern Time

Scan the QR code to download  
the Conversio Health app.

